Myriad partner laboratorium– MyChoice CDx Genome Instability Score Version 3

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Selecting an optimal sample

How should samples be prioritized?

1. Chemotherapy-naïve tumors from primary debulking surgery or biopsy
2. Chemotherapy-treated tumors from primary debulking surgery or biopsy
3. Cytology cell blocks (e.g., ascites fluid) are acceptable BUT tumor content must exceed 30%

What type of samples should be taken?

MyChoice GIS can use any subtype of invasive ovarian, fallopian tube, and primary peritoneal cancer.**

**Omentum and lymph node are accepted but are not ideal for successful testing.

How to select an optimal block:

A tissue block is preferred over slides when available:

- At least one tumor block with a cross-sectional area $\geq 25\text{mm}^2$ that contains at least 40 μm of tumor



If only slides are available, follow these preparation instructions:

- Cut and label one 5 μm section for H&E staining on a charged slide
- Cut and label 5 μm sections on uncharged slides according to the table below

Area of tissue (mm ²) with tumor $\geq 30\%$	Number of slides
20-25	8
15-19	12
10-14	16
5-9	20

- Slides should be numbered in order of cutting and display a specimen identification number
- If submitting a block, include at least an equivalent amount of tissue remaining as listed in the table

All sample specimens must contain > 30% tumor content by pathologic review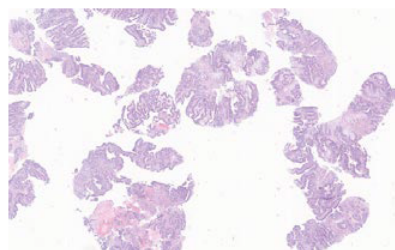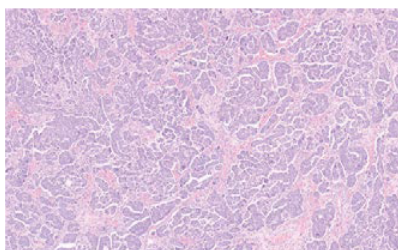



These examples illustrate high tumor percentage in comparison to normal tissue

Low-quality sample selection affects patient outcomes

Myriad Genetics understands sample selection cannot always be optimal, but MyChoice GIS can still give the results each woman needs with enough tumor content.

Ensuring specimens have at least 30% tumor cells in tissue or fluid samples is the best way to ensure MyChoice GIS success.

Which sample features are low quality?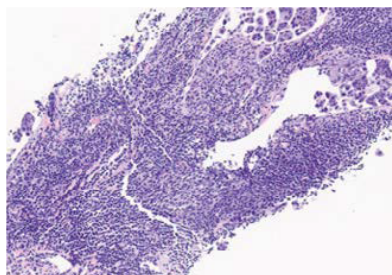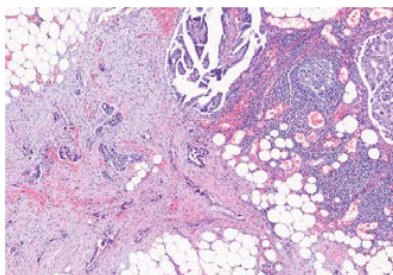

These examples illustrate omentum with stromal response and marked inflammation (left) and lymph nodes with lymphocytes that greatly outnumber tumor cells (right)

Which samples are unacceptable?

- Brain metastases
- Bone metastases treated with acid decalcification
- Endometrial primary tissue
- Extracted DNA

Best practices for tissue handling

	Parameter	Process
Collection	Cold ischemic time	Tissue should be put into fixative as quickly as possible
	Fixative ratio	Minimum 10:1 fixative to tissue
	Fixation time	< 72 hours
	Temperature	Avoid high temperatures that will degrade DNA
Processing	Processor chemicals	Use high-quality reagents and replace regularly
	Dehydration	Full dehydration is important for long-term DNA preservation
Embedding	Wax	Use high-quality, low-melting point paraffin wax that does not contain additives or beeswax
Shipping	Temperature	Ice packs are recommended to avoid warping of the block and DNA degradation

High-quality sample selection means high-quality patient care